OASIS® Iris Expander

DESCRIPTION:

The OASIS® Iris Expander is a square, one-piece blue polypropylene ring with a pocket at each corner for capturing and supporting the iris in a dilated condition. The Iris Expander is packaged within a clear holder along with an inserter for removing the Iris Expander from the holder and delivering it through a small incision into the eye.

CAUTION: This product is sold in a sterile condition and is intended for single use. Do not attempt to re-sterilize this device as it may damage the polypropylene material.

INTENDED USE:

The OASIS® Iris Expander is used to temporarily expand and support the iris in a dilated condition to aid in performing ophthalmic surgery through the pupillary opening.

INSTRUCTIONS:

The OASIS® Iris Expander is a delicate device, handle with care. If the Iris Expander appears to be damaged or defective do not use it.

LOADING THE OASIS® IRIS EXPANDER INTO THE INSERTER

Note: Do not load the Iris Expander into the inserter until ready to use.

1. The inserter comes pre-attached to the Iris Expander inside the holder. Use care in handling.
2. Remove the inserter/holder assembly from the package.
3. SLOWLY pull the inserter half out of the holder. This moves the Iris Expander into the folding zone of the holder.
4. SLOWLY move the thumb button back, which draws the folded Iris Expander into the inserter. Continue until the Iris Expander is fully into the inserter tube.
5. Remove the inserter from the holder and discard the holder.


INTRODUCING THE OASIS® IRIS EXPANDER INTO THE EYE


1. Fill the anterior chamber of the eye with viscoelastic.
2. Advance the tip of the inserter tube up to or into the anterior chamber.
3. Move the thumb button forward to push the Iris Expander from the inserter tube so that it rests on the anterior surface of the iris.

NOTE: The Iris Expander will appear slightly elongated and may have a bent crossbar after it has been expelled from the inserter. This will not affect its performance.

4. Use a Sinsky II or similar style hook to engage the pockets at each corner of the Iris Expander. Push the pocket into the pupillary space, engage the iris margin in the pocket, and release.
5. Place one proximal pocket first, then place the opposing distal pocket. Then place the other proximal pocket followed by the remaining distal pocket

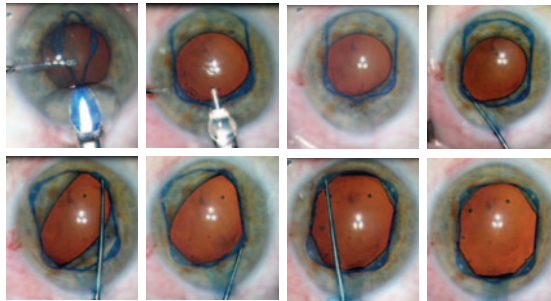
6. **ALTERNATIVE METHOD:** Place the left proximal pocket followed sequentially counter-clockwise by the right proximal, right distal, and left distal pockets.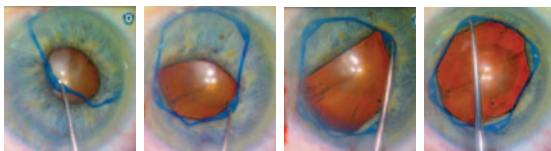


REMOVING THE OASIS® IRIS EXPANDER FROM THE EYE

1. Use a collar button, Kuglen or similar style hook to engage middle of the proximal crossbar.
2. Push toward the center of the pupil. This simultaneously disengages both proximal corner pockets.
3. Then place the same hook underneath the proximal crossbar and pull the Iris Expander through the wound.
4. Discard the Iris Expander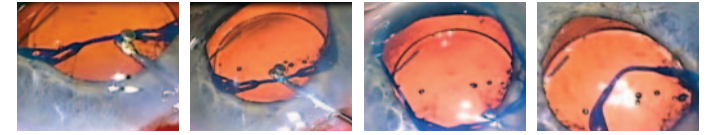

ALTERNATIVE: REMOVING THE OASIS® IRIS EXPANDER FROM THE EYE

1. Use a Sinsky II or similar style hook to engage a proximal pocket.
2. Disengage the pocket from the pupillary margin by pushing the pocket into the pupillary space and lifting it over the anterior iris.
3. Disengage the second proximal pocket in a similar manner.
4. Pull the Iris Expander from the eye using the Kuglen style or other push/pull hook. This will automatically disengage the other pockets from the iris margin.
5. Discard the Iris Expander.

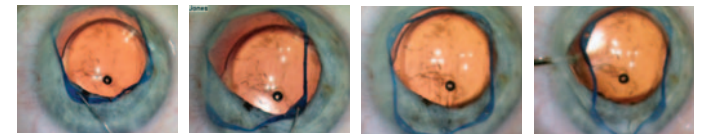CAUTION: The OASIS® Iris Expander is intended for single use. Do not re-use this product. The product cannot be properly cleaned and re-sterilized. Federal law (USA) restricts this device to sale or use by or on the order of a physician.

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